IEC 61000-4-6:2008
IEC 61000-4-8:2009
IEC 61000-4-11:2004

VIZSGÁLATI LABORATÓRIUM

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QuietekTELS

Hongye Rd., Suzhou Industrial Park Loufeng Hi-Tech Development Zone., Suzhou, Kína
86-512-62515088 FAX:+ 86-512-62515098 E-mail: service@quietek.com <http://www.quietek.com>

AC-4000 Digital Colposcope with software

FEATURE AND BENEFITS

1. **30X Optical magnification**
2. **16X Digital magnification**
3. **LED Coaxial illumination**
4. **Mini LCD display screen on the back of the head of colposcope**
5. **Abundant image data base of cases (up to 175 kinds), convenient for the doctors to make a comparative analysis.**
6. **Advanced image processing technology, can achieve the functions of marking, calculation of areas, zooming images.**
7. **With the editing functions of accessing images, freezing, collecting and deleting images, etc.**
8. **Images can be full-screen displayed, collected continuously and displayed on the screen at the same time. It is convenient to contrast.**
9. **The reports can be printed into multiple formats.**
10. **For languages of software, English, Spanish, Vietnamese and Chinese.**

SPECIFICATION & STANDARD OUTFITS

Camera	1/2.8" SONY CCD	1/2.8" COLOR CCD
2Pixels	3,270,000pixels	2,000,000 pixels
Resolution	1100 TV lines	700 TV lines
E-flip	yes	
Mirror image	yes	
Electronic Shutter	automatic	
S/N ratio	50db	
Illumination	super bright high color index shadows less LED light	
Color temperature	5500K	
Test timer	yes	
Multi-grade green filter	yes 4 grades	
Blue filter	no	
Focus	automatic and manual	
Focus distance	20-40cm	
Viewing area	Φ16-480mm	
Depth of view	4.3mm-129mm	
Magnification display	yes	
Average LED lamp life	≥50,000 hours	
Video output	BNC	
Power supply	100-240VAC,50/60Hz, Made in China	

POWERFUL AND COLORFUL COLPOSCOPE


1. Our software offers following function in total inspection

- 1) Course for displaying, freezing, capturing, deleting image, etc.
- 2) Methods of taking images and collecting 1-99 pieces of color images for every inspection.
- 3) Take images by foot switch and roll browse images.
- 4) It can be used in colposcope inspection and dynamic observation of operation treatment.
- 5) Unique acetic acid test timer function, real-time display of acetic iodine reaction testes.
- 6) It has white balance and green filter (as Pic.1)

Pic. 1


2. The software has abundant 175 standard atlas, helpful to contrast, analyze and diagnose. Tree-like atlas list to find aim atlas quickly, can save special case atlas for subsequent us. (as Pic. 2)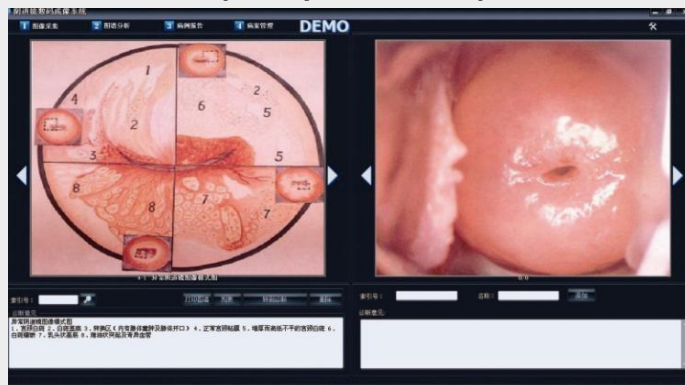

Pic. 2

3. The software can mark on the pathological changes image, add remark, measure the pathological change size, and magnify partially or totally. (as Pic. 3)

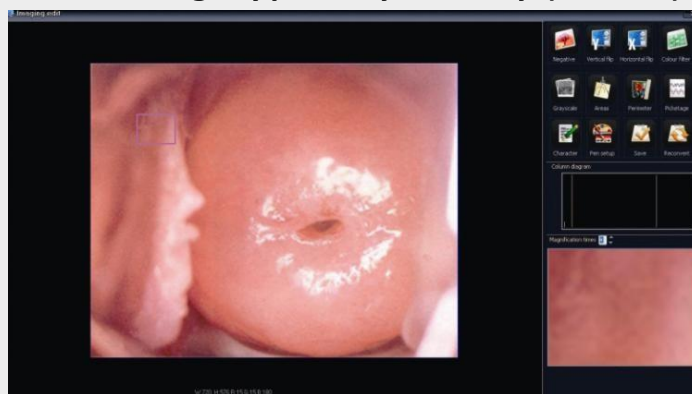
Pic. 3

4. It can input clinic symptom, clinic handling advice and so on. It can print 11 kinds of reports. RCI evaluation function. (as Pic. 4&5)

Pic. 4

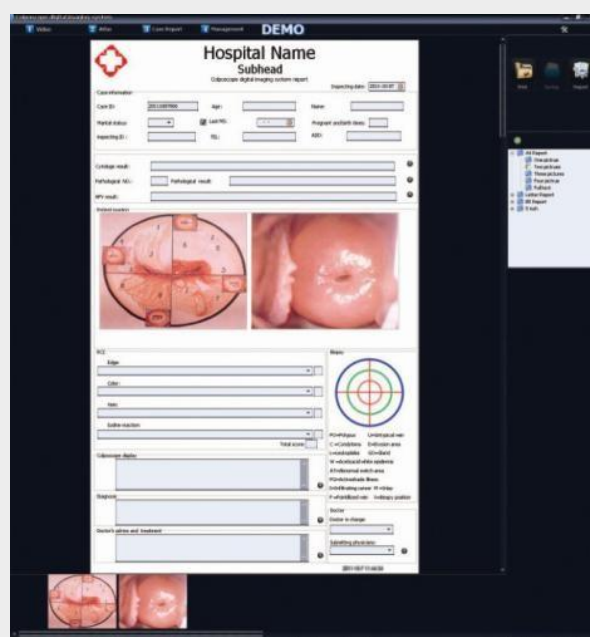Pic.5

5. It offers many functions to the cases information, such as adding, deleting, editing, memorizing, reviewing and so on. It can be searched by case number, name, age and date. It also supports blurry search. It can display cases in image after search, edit the data, memorize, print, magnify the image and support re-diagnosis. (as Pic. 6)

Pic. 6

Items and specification subject to change, please contact us for current product information.

ALLTION[®]

MICROSCOPES ARE YOUR BUSINESS & SERVICES OUR PRIDE

Alltion (Guangxi) Instrument Co.,Ltd

Alltion (Wuzhou) Co.,Ltd

Alltion Building, No.10, 3rd Road, Wuzhou Industrial Park, Wuzhou, Guangxi, China. 543000

Tel: +86-0774-2836101, 2836102

Fax: +86-0774-2836192

Email: sales@alltion.com or sales@alltion-microscope.com

Web site: www.alltion.com or www.alltion-microscope.com

AC-4000 Digital Colposcope with software

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Alltion (Guangxi) Instrument Co., Ltd
Alltion (Wuzhou) Co., Ltd